## Document Name and Number:

D-730-02 Sterile Disposal Skin Marker IFU (Radiation Sterilization)\_V1

Document information, authorship, and approvals			
<b>Author signs to confirm technical content</b>			
Prepared by: Angela Huo <i>Angela</i>	Job title: External Consultant	Signature:	Date: 28/2/2024
<b>Subject matter expert reviewer signs to confirm technical content</b>			
Reviewed by: Frankie Ng <i>Frankie</i>	Job title: Managing Director	Signature:	Date: 28/2/2024

Document history			
Revision No.	Revision Date	Description of change	Author
V1	28/2/2024	Initial Release	Angela Huo


**Sterile Disposable Skin Marker**  
**Marcador de piel desechable estéril**  
**Marcqueur cutané stérile jetable**  
**Pennarello cutaneo monouso sterile**  
**Marcador de pele descartável estéril**  
**Steriler Hautmarkierstift**  
**Stecil hudmarkör för engångsbruk**  
**Steni Tek Kullanımlık Cilt İşaretleyci**

1. English  
 2. Español  
 3. Français  
 4. Italian  
 5. Portuguese  
 6. Deutsch  
 7. Swedish  
 8. Turkish

Primary structure / Estructura primaria / Structure primaire  
 Struttura primaria / Estructura primaria / primäre Struktur  
 Primär struktur / Birincil yapı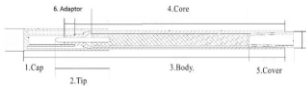

Dongguan Concord Medical Products Ltd.  
 Floor 4, Block C, Yue Neng Ind Park  
 Xia Bian Village, Houjie, Dongguan  
 China 523960  
 Tel : +86-769-88919066 Fax : 88919050  
[www.concord-medical.com](http://www.concord-medical.com)

EC MEP  
 med Europa GmbH  
 Langenhagener Str. 71  
 30855 Langenhagen

Document No. D-730-02  
 Version 1 Initial Release  
 Issue Date 28/2/2024

Symbols and label descriptions  
 Símbolos e descrições de etiquetas  
 Symbole und Beschreibungen der Etiketten

**REF** Item Number  
 número de artículo  
 numéro d'article  
 codice articolo  
 número de item  
 Katalognummer  
 Artikelnummer  
 Ürün numarası

**LOT** lot number  
 número de lote  
 numéro de lot  
 numero di lotto  
 número de lote  
 Chargennummer  
 Partinummer  
 Lot Numarası

**M** Date of manufacture  
 Fecha de manufactura  
 Date de fabrication  
 Data di produzione  
 Data de fabricaço  
 Herstellungsdatum  
 Tillsverkningsdatum  
 Üretim tarihi

**U** Use by date  
 Utilizar por fecha  
 Utiliser par date  
 Usare entro la data  
 Usar por data  
 Verwendbar bis  
 Använd senast datum  
 Tarihi Ötre Kullan

**S** Keep away from sunlight  
 Mantener alejado de la luz solar  
 Tenir à l'écart du soleil  
 Tenere lontano dalla luce solare  
 Manterha longe da luz solar  
 Vor Sonnenlicht schützen  
 Håll borta från solljus  
 Güneş ışığından uzak tutun

**W** Do not use if package is damaged and consult instructions for use  
 No utilizar si el paquete está dañado y consultar las instrucciones de uso.  
 Ne pas utiliser si l'emballage est endommagé et consulter les instructions d'utilisation  
 Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso  
 Não use se a embalagem estiver danificada e consulte as instruções de uso  
 Nicht verwenden, wenn die Verpackung beschädigt ist, und Gebrauchsanweisung beachten  
 Använd inte om förpackningen är skadad och se bruksanvisningen  
 Paket hasarlıysa kullanmayın ve kullanm talimatlarına bakın.

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## **ENGLISH** Sterile Disposable Skin Marker

### **Description**

Skin marker is a medical pen used for marking on the patient's skin before or during surgical procedures

### **Specifications**

Regular tip, Regular tip w/ ruler, Regular tip w/ ruler and 6 labels

Fine tip, Fine tip w/ruler, Fine tip w/ ruler and 6 labels

### **Characteristics**

Skin marker is used in marking and selecting the surgical sites before or during surgical procedures. Long-lasting scrub-resistant, practically odourless ink, contains primarily gentian violet, which is non-toxic, non-allergenic and non-irritating. Single-use, radiation sterilized product that will not come off easily during the disinfection procedure with 75% alcohol and iodophors. Quick, easy, and convenient to use in general surgery, plastic surgery, orthopaedic surgery, neurosurgery, cardiovascular surgery, and radiation therapy.

### **Product construction**

Skin marker is composed of 1. Cap, 2. Tip, 3. Body (Barrel), 4. Core, 5. Cover (End Cap), 6. Adaptor. Optionally with Ruler and 6 labels.

### **Intended use**

Skin marker and selection of the surgical site for diagnostic and interventional radiology, surgery, medical teaching and treatment of skin diseases.

### **Directions for use**

1. Sterilize the skin according to the standard procedure.
2. Mark the skin and wait for the ink to dry (about 6 seconds).
3. Further sterilization is not required, or sterilize again with iodine.
4. Test the marker on a small area of intact skin to ensure it is working properly and to check for any allergic reactions.
5. Marking can be easily removed with Chlorhexidine Gluconate.

### **Warnings**

1. Consider the possibility that the patient may be allergic to Gentian Violet. If patient feels uncomfortable while using, stop using and treat appropriately.
2. Radiation sterilized, for single use only, do not reuse or re-sterilize. The reuse may cause cross contamination which leads to microbial infection.
3. Should be used on dry skin. If using on sweating or wet skin, please wipe dry and operate as above Directions for use.
4. Protect the tip during use and cover with cap when finish using.
5. It should be used immediately after opening the package and disposed of safely and properly as medical rubbish after use.
6. Do not use if package or product has been damaged or soiled.
7. DO NOT ingest. In case of accidental ingestion, DO NOT induce vomiting. Rinse mouth thoroughly with water and seek for medical advice.
8. Avoid contact with eyes. In case of accidental contact, flush eyes with water and seek medical advice.

### **Contraindication**

Use on mucosa and open wound is not recommended.

**Possible complications**

None

**Storage**

Store in a cool, dry place. Avoid exposure to water, direct sunlight extreme temperature, high humidity or corrosive gases.

Temperature limit 5C° - 25C°

Sterile for 3 years. Expiry date : see package

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